PAPILOCARE®

Gel vaginal

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PAPILOCARE®

Vaginal gel

Gel for the prevention and adjuvant treatment of HPV-induced lesions

EN

WHAT IS PAPILOCARE® USED FOR?

Papilocare® is indicated for:

- Controlling and helping re-epithelialise the cervical transformation zone to prevent the risk of HPV-induced lesions (LSIL)
- Adjuvant treatment of HPV-induced intra-epithelial lesions
- Repairing and helping re-epithelialise lesions of the cervical-vaginal mucosa
- Treating cervical-vaginal mucosal dryness
- Rebalancing vaginal microbiota
- Improving vaginal health
- Creating the conditions for rapid healing of lesions caused by scratching due to burning or itching
- Forming a protective film that rapidly reduces irritation, creating suitable conditions to promote the natural healing process

HOW DOES PAPILOCARE® WORK?

Papilocare® is a gel that acts by forming a film on the cervical mucosa, creating a defensive barrier to prevent the risk of HPV integration. In this way, it disrupts HPV integration, preventing the formation of new lesions, and also acts as to aid in re-epithelialising pre-existing lesions. It also stimulates the regeneration of the vaginal microbiota.

WHAT DOES PAPILOCARE® CONTAIN?

The main ingredients of the formulation are:

- Components that allow the formation of a mucoadhesive gel on the cervical-vaginal mucosa, creating a protective film on it that promotes the product’s barrier action.
- **Hyaluronic acid niosomes**, with hydrating action, to help maintain tissue elasticity.
- **β-glucan niosomes**: whose antioxidant properties maintain the structure and natural functionality of the skin and mucous membranes.
- ***Centella asiatica* phytosomes**, which act to repair mucosal lesions.
- ***Coriolus versicolor* extract**, with re-epithelialising and restorative action for cervical-vaginal lesions and microlesions.
- **Neem extract**, with emollient properties that promote the natural healing process.
- **BioEcolia®**, a prebiotic that stimulates the growth of beneficial bacterial flora and rebalances the microbiota in the cervical-vaginal zone.
- ***Aloe vera* extract**, with hydrating, re-epithelialising, and restorative action for the cervical-vaginal zone.

WHO CAN USE PAPILOCARE®?

The use of Papilocare® is recommended in women (over 18 years of age) infected by the human papillomavirus (HPV), independently of whether they have virus-induced lesions (ASCUS or LSIL).

MODE OF USE

The treatment duration is 6 months, which can be extended for longer based on the medical prescription. It is important to complete the full length of treatment in order to obtain the desired results.

It is recommended to start treatment after menstruation.

First month: One cannula should be applied per day, for 21 consecutive days, then taking the next 7 days off.

Following months (month 2 to 6): One cannula should be applied every other day for 21 days.

The recommendation to pause treatment with Papilocare® during menstruation has to do with patient comfort, and not because there is any clinical incompatibility.

Papilocare® should ideally be used just before bedtime. Papilocare® is compatible with the use of condoms.

PAPILOCARE®

Vaginal gel

Gel for the prevention and adjuvant treatment of HPV-induced lesions

HOW TO APPLY PAPILOCARE®

Papilocare® is supplied in single-use vaginal cannulas. Any cannula that appears open or broken should be discarded. The cannula is designed to aid in the product’s application. For application, lay on your back. The cannula should be inserted into the vagina, then press to release its contents. Then, remove it from the vagina, while continuing to press the cannula, in order to not suction out the product already applied.

It is possible that a small amount of gel remains in the cannula. This does not mean that a dose less than that recommended has been administered, as the cannula releases a sufficient amount to cover the cervical-vaginal mucosa (3 ml). Once the cannula is used, it should be discarded. A cannula should not be reused nor shared between two patients.

WARNINGS

Papilocare® should not be used in people who have known hypersensitivity to any of its ingredients.

If after the application, you feel discomfort, discontinue the use of the product and consult a doctor.

There is no available data on the use of Papilocare® in patients using vaginal contraceptives (for example, vaginal ring, female condom). Therefore, Papilocare® is not recommended for use in women using these types of contraceptives. In case of pregnancy, a doctor should be consulted prior to using Papilocare®, and its use must only be considered under strict medical supervision.

ADVERSE REACTIONS

The safety profile of Papilocare® is good and no serious adverse reactions have been observed in the clinical studies performed with the product or since its marketing.

The only adverse reaction reported since the product has been marketed is irritation following application, with a frequency not exceeding one case per 10,000. This reaction is more common in women with a very sensitive or significantly damaged genital area or in post-menopausal patients with a high vaginal pH (pH 7). If this occurs, space out the doses (every 3 days) and consult your doctor.

CLINICAL EVIDENCE

Several studies have been carried out with Papilocare® that support its clinical efficacy and safety.

In several studies performed by the manufacturer, improvement in cervical-vaginal mucosal re-epithelialisation was observed with the use of Papilocare®. In a study comparing mucosal repair in untreated women and women treated with Papilocare®, it was observed that in women using Papilocare®, the percentage of women whose HPV-induced lesions were repaired was higher than for untreated women.

Other studies performed by independent investigators in university hospitals also confirmed the efficacy and safety of use of Papilocare®.

PAPILOCARE® STORAGE

Papilocare® should be kept between 2°C and 25°C. Keep out of reach of children.

PRESENTATION

Papilocare® vaginal gel is presented in the following formats:

- Package with 7 single-dose cannulas
- Package with 21 single-dose cannulas
- Package with 2 single-dose cannulas (sample format, non-commercial)
- Package with 1 single-dose cannula (sample format, non-commercial)

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0051

Procare
Health
Naturally woman

Distributed in UK:
PROCARE WOMEN HEALTH LTD.
271 High Street Berkhamsted Herts
HP411A. England