PALOMACARE®

Gel vaginal

ES

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Vaginal gel

EN

Moisturizing and repairing gel for the cervical-vaginal mucosa

Palomacare® Vaginal Gel has the following properties:

- Hydrates and restores the epithelium of the cervical-vaginal mucosa
- Re-balances the vaginal microbiota
- Improves vaginal health

Its gel formulation guarantees that the product remains in contact with the vaginal mucosa for longer periods of time. In this way, the vaginal mucosa recovers its normal elasticity and hydration, promoting physiological repair processes.

Recommended use: Medical device useful for:

Situations that affect vaginal health, with the presence of dryness and/or vaginal atrophy:

- Menopause, post-partum and breastfeeding
- Post-gynaecological surgery: conisation, LETZ (loop excision), cervical biopsies, vaginal surgery
- Radiation therapy and chemotherapy

Situations that affect vaginal health by altering the vaginal microbiota:

- Broad-spectrum antibiotic therapy
- Situations of suppressed immunity
- Poorly controlled diabetes
- Chronic vaginal infections

Mechanism of action: Palomacare® Vaginal Gel is an innovative, non-hormonal formulation whose primary action is to form a film on the vaginal mucosa that, combined with various natural ingredients in niosomal and phytosomal form, moisturizes with an improved comfort (by reducing the sensation of "losses" perceived by patients).

Instructions for use:

In acute situations: It is recommended to apply one cannula, once a day.

Mild situations and maintenance: It is recommended to apply one cannula every other day.

Apply a single-dose cannula in the vaginal area, preferably at bedtime. Press the applicator to empty its contents and remove gently. Do not rinse. Discard the single-dose cannula.

Primary actions of Palomacare® components; moisturizing and repairing gel for the vaginal mucosa:

• **Hyaluronic acid niosomes:** Their primary action is tissue surface hydration, providing elasticity to mucous membranes.

• **β -glucan niosomes:** Their properties maintain the structure and function of the vaginal mucosa.

• **Centella asiatica phytosomes:** Promote the vaginal mucosa's natural repair process.

• **BioEcolia®:** Prebiotic (α -glucan oligosaccharide) that stimulates the growth of beneficial bacterial flora and re-balances the vaginal microbiota.

• **Aloe vera:** Hydrating, epithelializing, and repairing actions for the vaginal mucosa.

Caution: Not for use in persons with known hypersensitivity to any of the ingredients. If after application you experience discomfort, discontinue use and consult a physician. If pregnant, consult with a physician prior to use.

Tested under gynaecological control

Fragrance-free. pH=5

Paraben-free

Favours sexual intercourse

Compatible with condoms

Presentation: 6 single-dose 5 ml cannulas

Store the product in a cool and dry place
Keep out of reach of children

Distributed in UK:
PROCARE WOMEN HEALTH LTD.
271 High Street Berkhamsted Herts
HP411A England

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Rev: 12/2020

Rev: 12/2020